


mamivac[®]

 MAMILLOMETER

Find the correct size of nipple shields

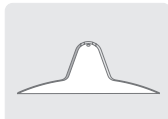

conical

S = 18 mm

M = 20 mm

L = 28 mm








cherry shaped

S = 18 mm

M = 22 mm



L = 28 mm

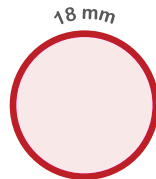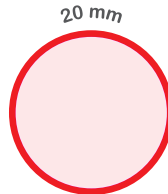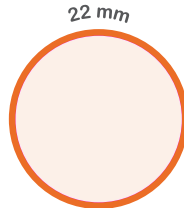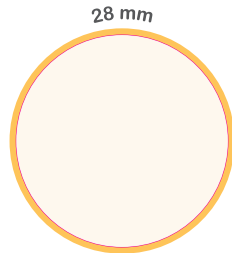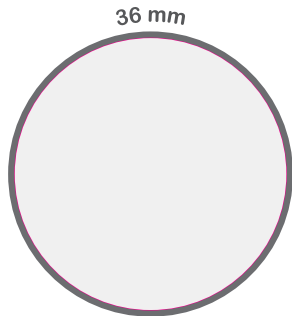

How do you find the correct size?

Place the different punched rings over your nipple and check which size you need.

- The size of the nipple shield depends on the mamilla and not on the areola.
- Take the measurement in a relaxed atmosphere.
If the nipple has about 2 mm of leeway, you have found the nipple shield with the correct fit.
- If you are not sure, ask your lactation consultant for support.

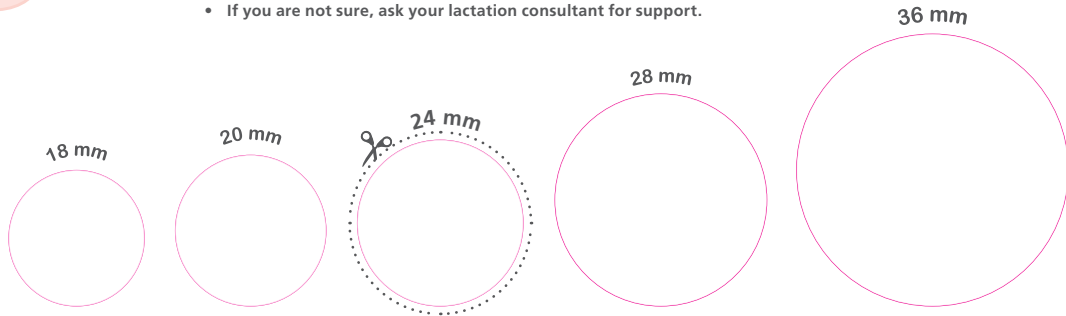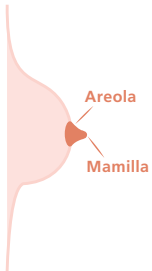

Our breast shield and suction kit sizes

